

APPENDIX: Installing CIRRUS HD-OCT Review Software

ZEISS Quick Help: CIRRUS™ HD-OCT

Technology Requirements

To successfully install and run CIRRUS™ HD-OCT Review Software, your computer system must meet specific hardware and software requirements. In particular, for newer versions of CIRRUS HD-OCT Review Software, your computer system must have a **64-bit operating system**. (See chart below for compatibility.)

Note: The CIRRUS HD-OCT instrument must be running the same version of software you are using for the exam lane review station. CIRRUS HD-OCT Review Software version 9.0 and greater cannot be installed on a computer that is also running ATLAS® 9000 Review Software or VISUPAC® Review Software. Contact ZEISS customer support at 1-800-341-6968 for further information.

CIRRUS HD-OCT Review Software version compatibility matrix for Windows operating systems.

Windows operating system vs. CIRRUS HD-OCT Review*

CIRRUS HD-OCT Review	Windows® XP	Windows 7 (32 & 64 bit)	Windows 8.1 (32 & 64 bit)	Windows 10 (64 bit)	Windows Server 2008 R2	Windows Server 2012 R2
4.0	■					
4.5	■					
5.0	■					
5.1	■					
5.2	■	■			■	
6.0	■	■			■	
6.5	■	■			■	
7.0	■	■			■	
7.5 / 8.0		■	■		■	
7.6 / 8.1		■	■		■	■
9.0		64-bit only	64-bit only		■	■
9.5.0		64-bit only	64-bit only		■	■
9.5.1		64-bit only	64-bit only	■	■	■
10.0		64-bit only	64-bit only	■	■	■

*Blank cell indicates cannot install. Square dot indicates acceptable for install.

SER-8992 Printed in the USA CZ-V/2017
 The contents of this reference guide may differ from the current status of the product in your country. Please contact your regional representative for more information.
 Subject to change in design and scope of delivery and as a result of ongoing technical development. CIRRUS, ATLAS and VISUPAC are either a trademarks or registered trademarks of Carl Zeiss Meditec, Inc. or other companies of the ZEISS Group in the United States and/or other countries.
 © 2017 Carl Zeiss Meditec, Inc. All copyrights reserved.