

How do I change the Baseline for a GPA on an HFA3?

ZEISS Quick Help: Humphrey® Field Analyzer 3 (HFA3)

To change the GPA Baseline on an HFA3 Instrument, follow these steps.

1. From the home screen, select the **patient's name**, the press **Reports**.

The screenshot shows the HFA3 software interface for a patient named 'SAMPLE 1, GPA', 85 years old. The interface includes a search bar, a list of patient samples, and a detailed view of the selected sample. The 'Reports' button is circled in red.

Date	Time	Eye	Parameters	View	GPA Reliability
7/27/2015	11:00 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/27/2015	10:59 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/27/2015	10:58 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/9/2015	2:16 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/9/2015	2:15 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		

2. Select one of the most recent right and/or left eye exams.

Select one or more tests to generate reports... Total Selected: 1

Date	Time	Eye	Parameters	View	GPA Reliability
7/27/2015	11:00 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/27/2015	10:59 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/27/2015	10:58 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/9/2015	2:16 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		
7/9/2015	2:15 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest		

3. In the window, select your **GPA report type** (for example, Full GPA). Press **Next**.

Select one report type to view

- SFA
- SFA Overview
- SFA GPA
- GPA Last Three Follow Up
- GPA Summary
- Full GPA
- Three In One

- You will now have the default GPA selection, which includes two Baseline tests and the additional Follow-up exams. The two Base Line tests will be the two earliest tests. You can manually change the selection of both Baseline and Follow-up tests.

Baseline Tests: Selected: SITA-Standard

Date	Time	Eye	Parameters
7/27/2015	10:59 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/27/2015	10:58 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/9/2015	2:16 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/9/2015	2:15 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest

Follow-up Tests: Total Selected: 3

Date	Time	Eye	Parameters
7/27/2015	11:00 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/27/2015	10:59 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/27/2015	10:58 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest

- To change the Baseline selection, **uncheck** current Baseline selected exam and then select another exam option from the Baseline list. The application will only offer available follow up exams based on the new modified Baseline. Once the correct Baseline is selected, select **Next**.

SAMPLE 1, GPA 85 years

Unknown ID 1931.0101.222F.837... Date of birth 1/1/1931

Baseline Tests: Selected: SITA-Standard

Date	Time	Eye	Parameters
7/27/2015	10:59 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/27/2015	10:58 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/9/2015	2:16 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest
7/9/2015	2:15 PM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest

Follow-up Tests: Total Selected: 1

Date	Time	Eye	Parameters
7/27/2015	11:00 AM	OD	SITA-Standard Central 24-2, W-W III ThresholdTest

Refer to the Humphrey Field Analyzer user manual *Instructions for Use* for safe and effective operation of the instrument.

