

How do I clean the fan filter on my CIRRUS HD-OCT?

ZEISS Quick Help: CIRRUS™ HD-OCT

It is important to periodically inspect the fan filter located under the removable cover on top of the instrument. The filter should be checked at least twice a year and more frequently in dusty environments. Neglecting to maintain the instrument's filter can result in dust build-up that can affect the instrument's internal temperature control.

Cleaning the fan filter

1. To access the fan filter, first detach the top cover. Simply push on the snap connector, (Figure 1) pull the cover open (Figure 2) and remove the filter (Figure 3 and 4).
2. Dust and loose dirt on the fan filter can be cleaned by shaking the filter by hand or if needed it can be washed using a mild liquid soap and water. Allow the filter to thoroughly dry before placing it back into the instrument.
3. Replace the top cover.

Replacing the fan filter

If needed, order a new fan filter by contacting ZEISS Customer Care at 1-800-341-6968 and ask for a CIRRUS HD-OCT fan filter.



Figure 1



Figure 2



Figure 3

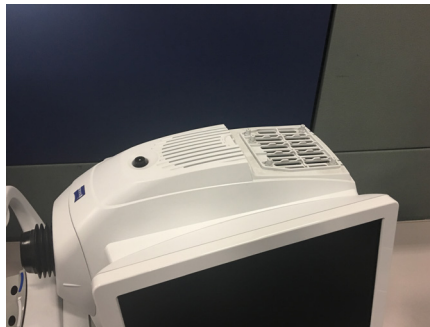


Figure 4

Refer to the CIRRUS HD-OCT user manual *Instructions for Use* for safe and effective operation of the instrument.



Carl Zeiss Meditec AG
Goeschwitzer Strasse 51-52
07745 Jena
Germany
www.zeiss.com/med



Carl Zeiss Meditec, Inc.
5160 Hacienda Drive
Dublin, CA 94568
USA
www.zeiss.com/med

