

# What does *ID* stand for in the upper left corner of the display on my VISULENS 500?

ZEISS Quick Help: VISULENS® 500

The **ID** in the upper left corner of your VISULENS 500 display screen is an identification (ID) counter that tracks the number of measurements taken (*Figure 1*).



Figure 1

If the counter is not preferred, it can be deactivated by changing this option on the settings menu.

Follow the steps below:

- Click the **ON** icon for the Settings Menu Screen.
- Set the ID display setting to **OFF**.

Refer to the VISULENS 500 user manual *Instructions for Use* for safe and effective operation of the instrument.