

## SL Imaging Solution cybersecurity update ("PrintNightmare")



Seeing beyond

# SL Imaging Solution cybersecurity update ("PrintNightmare")

Please note: This document does not replace the user manual which is delivered with the device.

## About the update

"PrintNightmare" is a vulnerability affecting Microsoft Windows operating systems. According to Microsoft, "a remote code execution vulnerability exists when the Windows Print Spooler service improperly performs privileged file operations. An attacker who successfully exploited this vulnerability could run arbitrary code with SYSTEM privileges. An attacker could then install programs; view, change, or delete data; or create new accounts with full user rights."

"PrintNightmare" does not affect the safety and performance of the ZEISS SL Imaging Solution. Nevertheless, ZEISS would like to offer you an update for your ZEISS SL Imaging Solution to fix the "PrintNightmare" vulnerability.


## Required storage media

You will need an empty USB flash drive with at least 1 GB of storage capacity to download the update files to prior to installation.


## Preparation

- Download the following files to the root directory of your USB flash drive:
  - LGPO.exe
  - PrintNightmare\_1.lgpo.txt
  - Install\_Patch\_PrinterNightmare\_CVE-2021-34527\_Win-10.cmd


# How to update your SL Imaging Solution

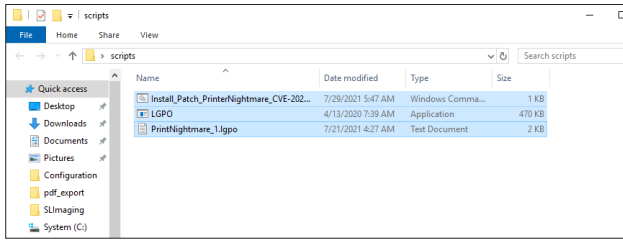


1. Power on the SL Workstation.
2. Connect the USB flash drive containing the update files.
3. Copy all three files to the following location  
c:\Users\zeiss\Desktop\scripts. Check that the .cmd file name contains "Win-10".

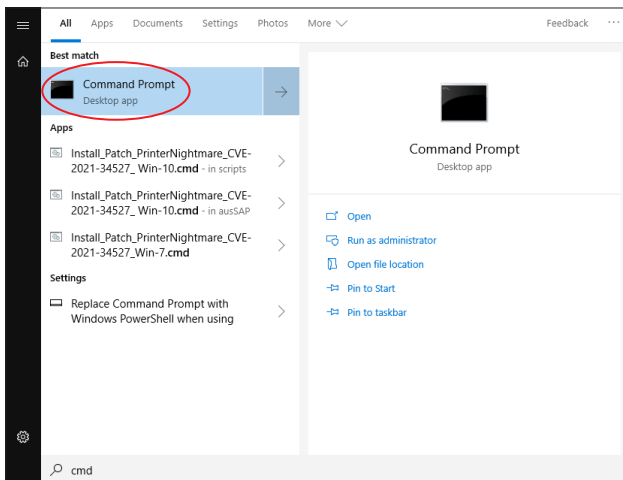


4. Click the start menu icon in the system taskbar and enter "cmd" in the search field. A view similar to the image on the left appears.

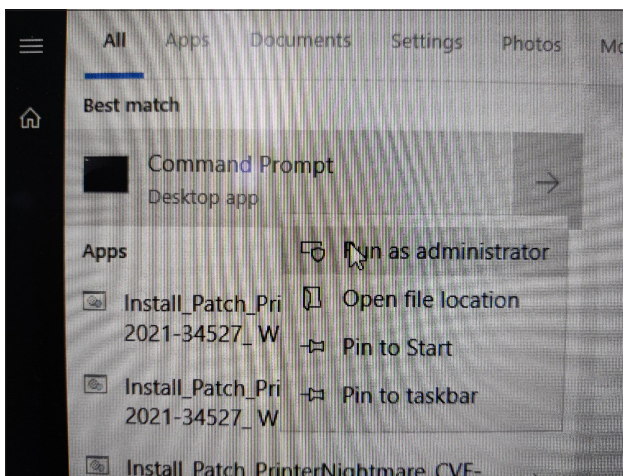


5. Right click on "cmd.exe" to open the context menu. Select "Run as administrator".

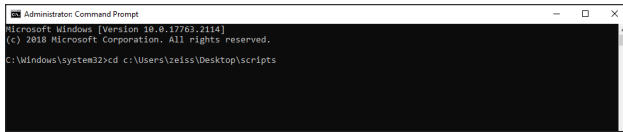


6. A command prompt with administrator rights opens.  
Enter "cd c:\Users\zeiss\Desktop\scripts" and press Enter.

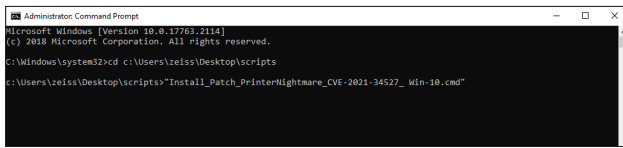


7. After the directory has been changed, enter "Install" and press the TAB key. The filename should be completed as shown in the image on the left. Press Enter to continue.

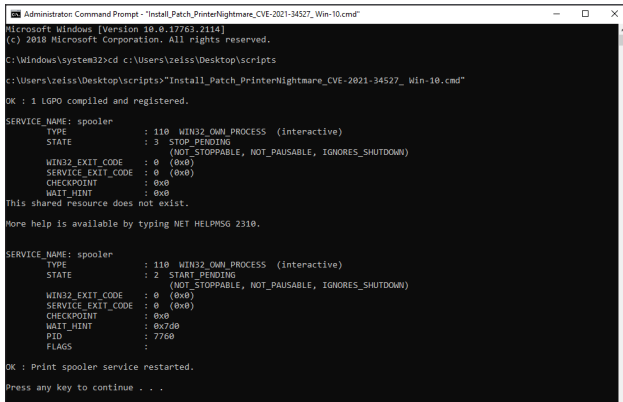

8. The script is executed.

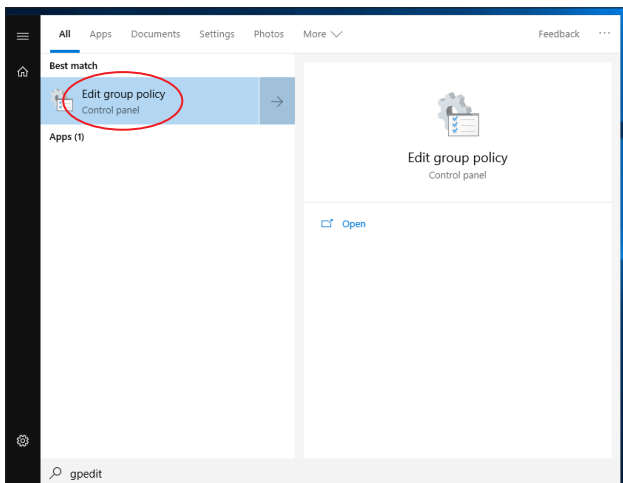

9. After the script has run successfully and completely, restart the computer for the changes to become effective.
10. After the restart, click the start menu icon in the taskbar. Enter "gpcedit". Click on the item that appears in the list to open the group policy settings.

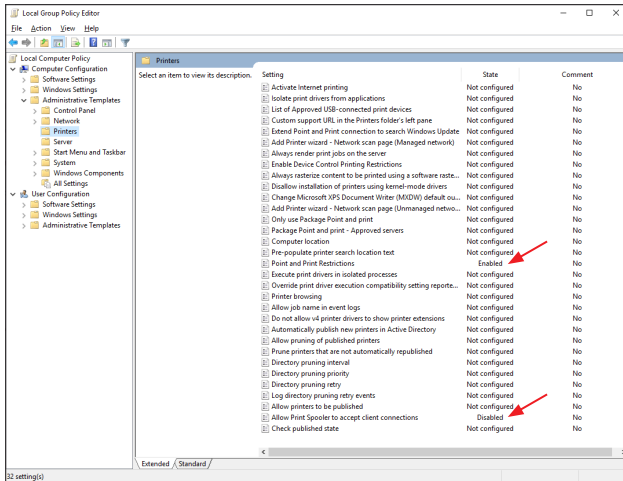

11. The group policy settings window opens. Go to “Computer Configuration” > “Administrative Templates” > “Printers”.

Check that the setting for “Point and Print Restrictions” is set to “Enabled” and “Allow Print Spooler to accept client connections” is set to “Disabled”.

Discover more expert videos, supporting documents, and common questions and answers in the ZEISS Product Insights.



ZEISS Product Insights website



**Carl Zeiss Meditec AG**

Goeschwitzer Strasse 51-52

07745 Jena, Germany

[www.zeiss.com/med](http://www.zeiss.com/med)

[info.meditec@zeiss.com](mailto:info.meditec@zeiss.com)

000000-2334-708-AddGA-GB-260821