

How do I connect HFA Review Software to an HFA3 instrument?

ZEISS Quick Help: Humphrey® Field Analyzer 3 (HFA3)

This document reviews the steps to connect HFA Review Software for use with an HFA3 instrument.

Software media package

To connect your HFA3 Software, you will use **HFA Software Media Package**, which contains two USB drives with everything you need to complete your software installation. The USB drives will be labeled **SW** [Software] and **UD** [User Document].

- HFA Review Installation Software is labeled “SW-R”



- HFA Review User Manual is labeled “UD-R”




Log on to HFA Review Software.

1. From the Desktop, double-click the HFA Review icon.
2. Once the ZEISS splash screen appears, followed by the Startup screen, click Continue.




3. The Login screen appears.

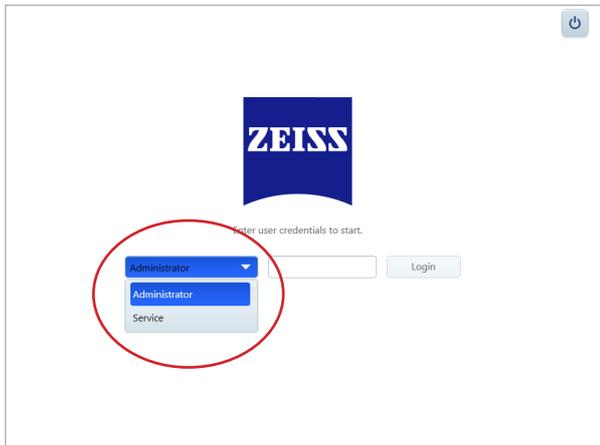

4. Select **Administrator** from the drop-down list, and type in the default password.

HFA Review software is designed to locate network-connected HFA3 instruments that are turned on, automatically establishing a connection with one of them. When this occurs, you will see the Patient Selection screen populated with patients. You are now connected and ready to use the HFA Review software.


Note: Default password: 0000 (ZEISS recommends that you change this password after logging in for the first time.)

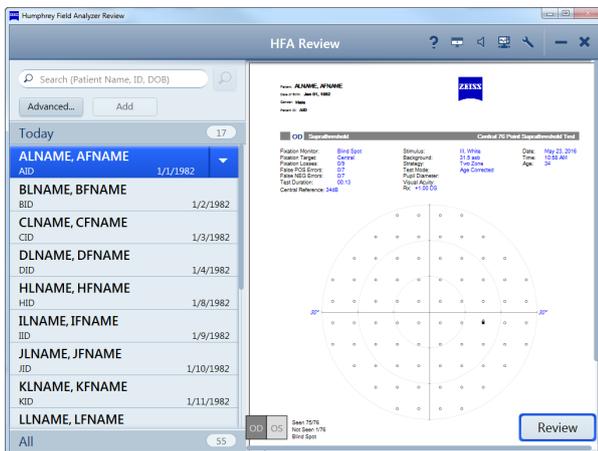

Connection Failure:

If the connection does not occur automatically, take the following steps:


1. Select the **wrench icon** from the HFA Review toolbar to go to Settings.
2. Select **Networking** and the Networking screen appears. Scroll down until you see the Available Instruments area.
3. Select one of the HFA3 instrument(s) from the Available Instruments list to which you want to be connected.
4. Click **Connect**. The red icon should turn green. HFA Review software is now connected and ready to use.

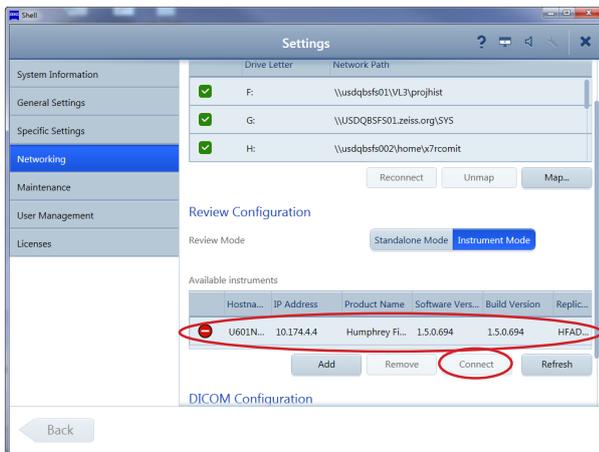


Manual Configuration:

If HFA Review software does not connect to an HFA3 instrument, go to "Manual Configuration":


1. From the Networking screen, click the **Add** button from the Available Instruments list.

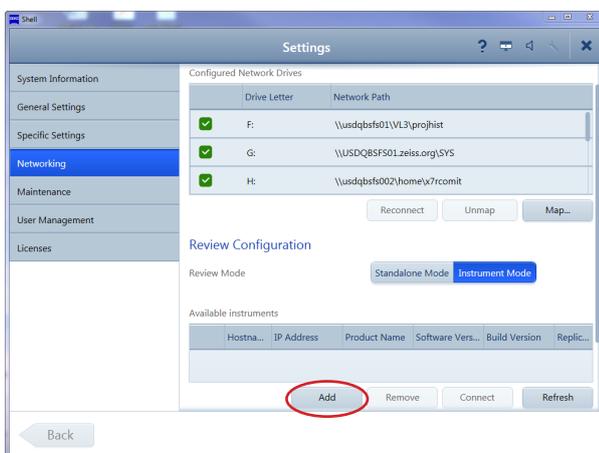

- The Host Name screen will appear. Type in the **Host-Name** or **IP address** of the HFA3 instrument. This can be found in the System Information area of the HFA3 instrument settings.
- Click **OK**. The instrument is added to the Available Instruments list.

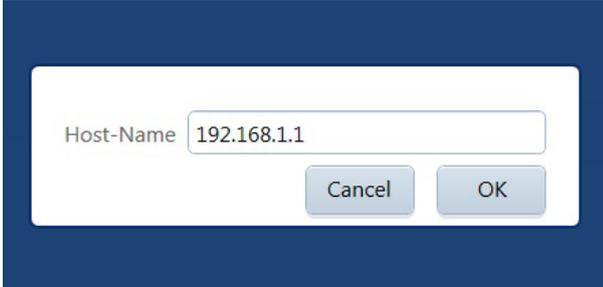

- Click on the instrument in the list and click Connect. The instrument will connect.

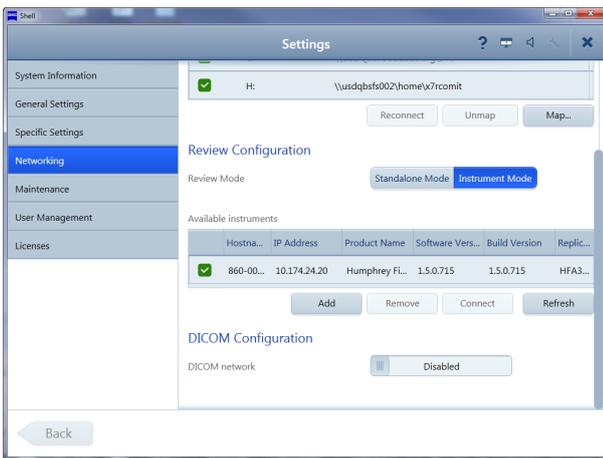

Note: If the instrument is not in the Available Instruments list, perform the following:

- Verify the IP configuration.
- Confirm that the HFA3 instrument and HFA Review application are in the same network, including the subnet components "A.B.C."
- Check the Firewall settings.

Refer to the HFA3 User Manual Instructions for Use for safe and effective operation of the instrument.



Carl Zeiss Meditec AG
Goeschwitzer Strasse 51–52
07745 Jena
Germany
www.meditec.zeiss.com/visuref100

Carl Zeiss Meditec, Inc.
5160 Hacienda Drive
Dublin, CA 94568
USA
www.zeiss.com/med/us

SER.10832 CZ-I/2019
The contents of this reference guide may differ from the current status of approval of the product in your country. Please contact your regional representative for more information. Subject to change in design and scope of delivery and as a result of ongoing technical development. Humphrey and HFA are either trademarks or registered trademarks of Carl Zeiss Meditec, Inc. in the United States and/or other countries. © 2019 Carl Zeiss Meditec, Inc. All copyrights reserved.