

Maintenance

Care and cleaning



WARNING - RISK OF ELECTRIC SHOCK

Prevent moisture from penetrating the device or keyboard. Disconnect the power cable from the power supply before cleaning or disinfecting the device.



CAUTION - RISK DUE TO CROSS-CONTAMINATION

Parts with which the patient has come into contact during the examination (chin rest, forehead rest) should be cleaned with a disinfectant approved for the purpose. These parts are resistant to cleaning agents categorized as "low" (e.g. suds, quaternary ammonium compounds) and "intermediate" (e.g. alcohol, Javel water, iodine); classification pursuant to: Disinfectants and activity spectrum according to the Center for Disease Control and Prevention, Atlanta, USA.

CAUTION - PROPERTY DAMAGE

The national disinfecting regulations must be observed in the choice of disinfectants and disinfection procedures. Please note that some cleaning agents and disinfectants may have an adverse effect on plastic components. Damage caused by such disinfectants is not covered by our warranty. The surfaces of the device have been tested and are guaranteed to resist frequent treatment with alcoholic disinfectants and cleaning agents in the long term.

Do not use aggressive or abrasive cleaning agents.

- Do not use acetone and acetone-based cleaning agents to clean the devices, as they could damage the surfaces.
- All parts of the housing may be wiped off with a moist but not drip-wet cloth. Wipe off any marks or stains with distilled water, to which a drop of household detergent has been added.
- Use conventional cleaning cloths for computers and monitors to wipe off the display and keyboard.
- Remove dust from optical surfaces by means of a fine brush.
- To protect from dust, when not in use, cover the device using the dust cover provided.