

Care of the device

The medical personnel are responsible for keeping the instruments and equipment in the best operating conditions. The simple steps described below constitute a practical guideline for defining a suitable care and maintenance program.

Cleaning

**CAUTION****Risk of infection!**

- Clean the front panel with a soft cloth dampened with distilled water. If necessary, use neutral detergent only.
- Do not use corrosive or abrasive cleaning agents.
- For cleaning please refer to the indications given in the relevant instructions for use.

Reusable surgical instruments must be cleaned and sterilized according to the indications given in the respective instructions for use.

Disinfection

It may be necessary to disinfect the surfaces.

NOTE**Damage to the surfaces on the instrument!**

- Use a disinfectant based on aldehyde and/or alcohol. The addition of quaternary compounds is acceptable. To prevent damaging surfaces, disinfecting agents other than those listed below must not be used.

The maximum concentrations are:

- For alcohol (tested with 2-propanol): 60%
- For aldehyde (tested with glutaraldehyde): 2%
- For quaternary compounds (tested with DDAC): 0.2%