

## Care of the device

The medical staff is responsible for keeping the medical device and further equipment in optimal operating condition. The simple steps described below constitute a practical guideline for defining a suitable care and maintenance program.

### Cleaning

**CAUTION****Risk of infection!**

- Clean the front panel with a soft cloth dampened with distilled water. If necessary, use neutral detergent only.
- Never use corrosive or abrasive cleaning agents.
- More information about cleaning can be found in the relevant instructions for use.

### Disinfection

It may be necessary to disinfect the surfaces.

**NOTE****Damage to the surfaces on the instrument!**

- Use a disinfectant based on aldehyde and/or alcohol. The addition of quaternary compounds is acceptable. To prevent damaging surfaces, disinfecting agents other than those listed below must not be used.

The maximum concentrations are:

- For alcohol (tested with 2-propanol): 60%
- For aldehyde (tested with glutaraldehyde): 2%
- For quaternary compounds (tested with DDAC): 0.2%