

# Recommended Protective Coverings For Your Ophthalmic Instruments

## ZEISS CLARUS 500/700



Seeing beyond

With the health and safety of you, your patients and your staff in mind during the COVID-19 pandemic, ophthalmic practices are advised to take extra precautions to prevent the spread of germs by using protective coverings on equipment when at all possible.



### Forehead Rest and Chinrest

Start with the forehead rest and chinrest cover.



### Handle Grips

Next, cover the handle grips.



### Mouse and Keyboard

To protect the operator of the device, cover the keyboard and mouse.

Using a single-use protective covering may help put your patient at ease and demonstrate your commitment to their health and well-being. Below, you'll find a list of **suggested** protective products to help facilitate a hygienic environment.

*These products are recommended to be used as single-use disposables and should be changed after each patient visit. Simply slip the sleeves on before the patient arrives, and remove and replace them before your next patient exam.*

Component	Barrier Type	Vendor*	Product*
Forehead Rest Chinrest	Polyethylene Film 4 x 6 in	Dental Office Products Noble Dental Supplies Dental Savings	D435-0002-CL 100-2102 BAR00-CLEAR
Mouse	Polyethylene Bag 4 x 8 in, 1 mil thick	U-Line	S-3450
Keyboard	Polyethylene Bag 7 x 18 in, 1 mil thick	U-Line	S-10877
Handle Grips	Polyethylene Bag 2 x 6 in, 2 mil thick	U-Line	S-1993

\*Suggested protective coverings are not sold with ZEISS products. Examples shown are select U.S. vendors, which may not be available in your market. You may find comparable vendors in your own market.

Please always refer to User Manual for any further instruction on operation of device.

EN\_31\_025\_0399II; CAM.12331 Rev B ©Carl Zeiss Meditec, Inc. 2020. All rights reserved.