VISUPAC 500 安全更新 ("PrintNightmare")

Seeing beyond

VISUPAC 500 安全更新 ("PrintNightmare")

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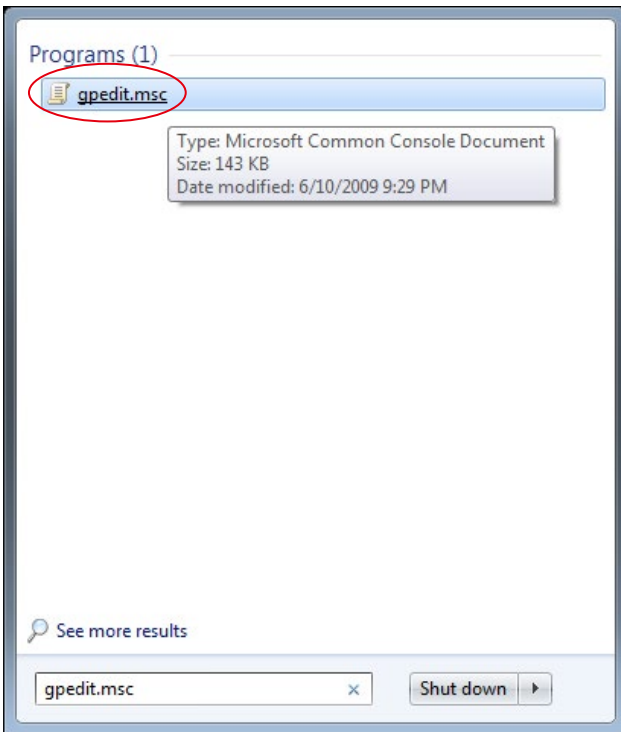
```
Administrator C:\Windows\System32\cmd.exe - Install_Patch_PrinterNightmare_CVE-2021-34527_Win-7.cmd
Microsoft Windows [Version 6.1.7601]
Copyright (c) 2009 Microsoft Corporation. All rights reserved.

C:\Windows\system32>cd C:\Users\Administrator\Desktop\scripts
C:\Users\Administrator\Desktop\scripts>Install_Patch_PrinterNightmare_CVE-2021-34527_Win-7.cmd
OK : 1 LGPO compiled and registered.
[SC] OpenService FAILED 1060:
The specified service does not exist as an installed service.

SERVICE_NAME: spooler
        TYPE               : 110  WIN32_OWN_PROCESS (interactive)
        STATE               : 1  STOP_PENDING
                        (NOT_STOPPABLE, NOT_PAUSABLE, IGNORES_SHUTDOWN)
        WIN32_EXIT_CODE      : 0  (0x0)
        SERVICE_EXIT_CODE  : 0  (0x0)
        CHECKPOINT         : 0x0
        WAIT_HINT           : 0x0
This shared resource does not exist.
More help is available by typing NET HELPMSG 2310.

SERVICE_NAME: spooler
        TYPE               : 110  WIN32_OWN_PROCESS (interactive)
        STATE               : 2  START_PENDING
                        (NOT_STOPPABLE, NOT_PAUSABLE, IGNORES_SHUTDOWN)
        WIN32_EXIT_CODE      : 0  (0x0)
        SERVICE_EXIT_CODE  : 0  (0x0)
        CHECKPOINT         : 0x0
        WAIT_HINT           : 0x740
        PID                : 3740
        FLAGS                :
[SC] StartService: OpenService FAILED 1060:
The specified service does not exist as an installed service.

OK : Print spooler service restarted.
Press any key to continue . . .
```

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Carl Zeiss Meditec AG