




## **CT LUCIA**

Handling instructions  
(EU Version)



Seeing beyond

# Preparing the CT LUCIA 621P/621PY and CT LUCIA 221P

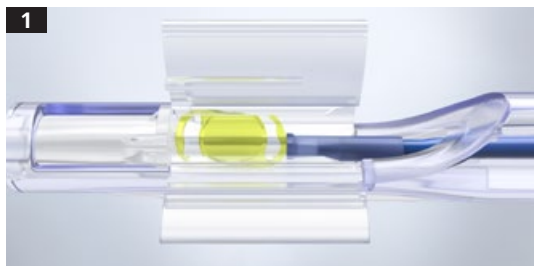


1 Verify the lens is central and secure in the IOL chamber



2 Cover the whole lens and blue plunger tip with a **generous** amount of OVD. Avoid touching the lens and blue plunger tip.




3 Close the lid of the IOL chamber. **IMPORTANT:** Allow the lens to remain in this position until the surgeon is ready to deliver it to the eye.

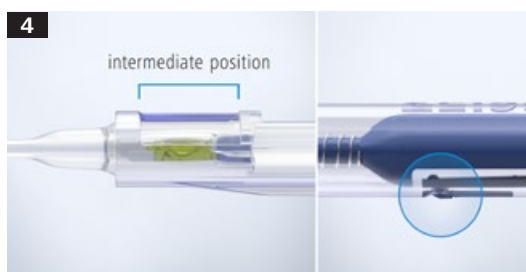

4 Advance the lens to the intermediate position. Gently press the plunger forward until an audible 'click' is heard.

**IMPORTANT:** The lens should be implanted immediately.

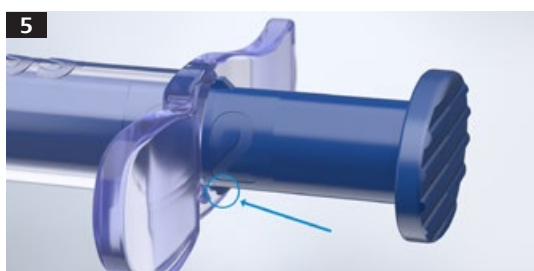

5 Slowly advance the lens until it has been released from the injector. If delivery is incomplete, apply additional pressure to the thumb flange to release the lens.

6 Carefully position the lens in the capsular bag.

7 Discard the device. Do not reuse the delivery system.

CE 0297

CT LUCIA 221P  
CT LUCIA 621P/621PY



Carl Zeiss Meditec AG  
Goeschwitzer Str. 51–52  
07745 Jena  
Germany  
[www.zeiss.com/lucia](http://www.zeiss.com/lucia)  
[www.zeiss.com/med/contacts](http://www.zeiss.com/med/contacts)

EN\_32\_022\_00781 Printed in Germany. CZ-1/2019 International edition: Only for sale in selected countries.  
The contents of the brochure may differ from the current status of approval of the product or service offering in your country. Please contact our regional representatives for more information. Subject to changes in design and scope of delivery and due to ongoing technical development. CT LUCIA is either a trademark or registered trademark of Carl Zeiss Meditec AG or other companies of the ZEISS Group in Germany and/or other countries.  
© Carl Zeiss Meditec AG, 2019. All rights reserved.